

GVB-geliMED E.Kfm  
Ginsterweg 4 a  
23795 Bad Segeberg

Tel.: 04551-956730  
Fax: 04551-956733  
e-mail: [info@gvb-gelimed.de](mailto:info@gvb-gelimed.de)  
<http://www.gvb-gelimed.de>

Rev.4/29.08.11

## **Connecting Lead Wire for disposable Disc-Electrodes**

### **Direction for use:**

Connect the cable with the disposable Disc-Electrode (1183AGCL10) checking for a good contact and connect plug into machine Head-Box.

### **Cleaning:**

clean the cable with a damp cloth and a mild detergent..Remains of conductive and salty fluids can be removed with a smooth brush.

**Take care never immerse the plugs in fluid.**

**Disinfection:** products normally used in hospitals for surfacedisinfection

**Storage:** 10 to +50 degrees

**Humidity:** 20 to 80%

### **Guarantee and Limitations:**

Producer guarantees this product is realised in compliance with ISO13485 standard and complies with 93/42/EEC directive.Producer declines any responsibility resulting from faulty operation of this product when used differently from direction of use.

**Waste disposal:** Waste coming from hospitals must be disposed of in accordance with regulation in force.

**Warning:** This cable is to be used only with the suitable disposable Disc-Electrodes.

### **ATTENTION:**

Cable without touchproofed 2/3/4mm plugs are only allowed to be used in accordance with the German Medizin-Product-rules DIN 60601/1.3.1996, for old EEG-Machines , to use it with their planed Headbox.